

# AimTab™ REDUCING SUBSTANCES TABLETS

AimTab™ Reducing Substances Tablets are for the semi-quantitative determination for the amount of reducing substances (generally glucose) in urine, stool, wine and other liquids.

**Quickly Test for  
Glucose, Lactose,  
Fructose, Galactose!**

Same test procedure as Clinitest®, a registered trademark used by Bayer HealthCare LLC.

## LOOK AT THESE GREAT FEATURES!

- ⇒ Quick & easy results for testing urine, stool, wine and other liquids.
- ⇒ Detects as little as 0.25 mg/dL of glucose.
- ⇒ Detects glucose and other reducing sugars such as lactose, fructose, galactose & pentoses.
- ⇒ Store at room temperature.



⇒ CAP / CLSI formatted procedures available via fax or email - just add your name!

## 5 DROP METHOD<sup>1</sup> (2 Drop Method also available)

- 1 ... With dropper in upright position, place **5 drops** of specimen in test tube. Rinse dropper with water and add 10 drops of water to test tube.
- 2 ... Drop **one tablet** into test tube. Watch while complete boiling reaction takes place. **DO NOT shake** test tube during boiling, or for the following 15 seconds after boiling has stopped.
- 3 ... At the **end of this 15-second waiting period**, shake test tube gently to mix contents. Compare color of liquid to Color Chart.
- 4 ... **Write down the percent (%) result** which appears on the color block that most closely matches the color of the liquid.



**POISON: CAUSES SEVERE BURNS. DO NOT SWALLOW!**

**THE TESTS YOU WANT.  
THE ACCURACY YOU NEED!**

**DISTRIBUTED BY:**



<sup>1</sup> See Package Insert for complete procedure.



**36 TABLETS  
PER BOTTLE**



**20 POUCHES PER BOX\***  
\*1 Tablet per sealed foil pouch



## TWO AimTab™ TABLET PACKAGING OPTIONS:

	SIZE	REF#
<b>BOTTLED TABLETS</b>	36 tests	13336
<b>POUCHED TABLETS</b>	20 tests	13320

## RELATED PRODUCT

	SIZE	REF#
<b>UA Liquid Controls</b>	2x5mL	51116

- ⇒ Level 1 (1x5mL)
- ⇒ Level 2 (1x5mL)
- ⇒ Used for Glucose control
- ⇒ Store at 2° to 8°C
- ⇒ Includes hCG



[www.germainelabs.com](http://www.germainelabs.com)

**GERMAINE**  
LABORATORIES, INC.



800.854.8446

210.692.4192



210.692.4198 Fax